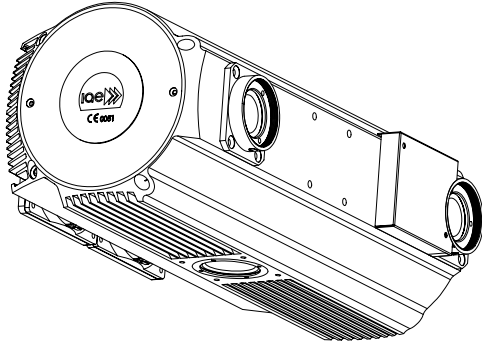
Documentazione guaina  
Tube assembly documentation  
Documentation gaine équipée

# C100 XT

Nr. di matricola  
Tube No.  
Nr de série

CE 0051

Questa documentazione deve essere fornita all'utilizzatore del complesso tubo guaina  
The contents of this documentation must be transmitted to the user of the tube-assembly  
Le contenu de cette documentation doit être transmis à l'utilisateur de la gaine équipée

Documentazione N° Documentation N° N° de Documentation	Revisione Edition Version	Data di edizione Date of release Date de l'édition	Testo originale Original text Texte original
C100XT	C	22.02.2022	italiano / italian / italien

---

## Sommario - Table of contents - Table des matières

---

Sommario - Table of contents - Table des matières .....	2
Descrizione - Description - Description.....	3
Etichettatura - Labeling - Etiquetage .....	5
Caratteristiche - Specifications - Spécifications.....	6
Curve di riscaldamento e raffreddamento della cuffia Tube-assembly heating and cooling curves Courbes d'échauffement et refroidissement de la gaine équipée.....	8
Dimensioni - Outline drawing - Schéma dimensionnel.....	10
Fissaggio meccanico / Mechanical mounting / Montage mécanique.....	12
Collegamenti elettrici / Electrical connections / Connexions électriques .....	14
Dati dello statore - Stator data - Données du stator .....	16
Informazioni sulla sicurezza elettrica - Information about electric safety - Information sur la sécurité électrique	17
Informazioni sulla sicurezza termica / Information about thermal safety / Information sur la sécurité thermique	18
Modifiche alla filtrazione - Modification of filtration - Modification de la filtration .....	19
Accessori - Accessories - Accessoires .....	20
Simbologia - Symbols - Symboles .....	21
Posizione macchie focali e rotazione anodo Focal spots position and anode rotation Position des foyers et rotation d'anode .....	21

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### **Declaration of conformity**

This tube fulfils the essential requirements of the regulation EU 2017/745 according to standard IEC 60613, IEC 60336, IEC 60522, IEC 60526, IEC 60601-1, IEC 60601-1-3, IEC 60601-2-28.

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Federal standard

### Material

Lead lined aluminium body. Filled under vacuum with insulating oil, specially processed. Fitted with an internal oil circulation system to improve temperature uniformity.

### Finish

Metallic grey.  
Other colours by request.

### Compatibility

Suitable for inserts with max. anode diameter of 110 mm.

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## **Intended use**

X-ray units produced by IAE are intended for use in medical diagnostics, in conjunction with equipment having the same purpose.

They are intended to generate X-ray used in radiological equipment with diagnostic functions.

They are not designed to be used in sterile environments, outdoors or in presence of explosive atmospheres.

## **Improper use**

The X-ray tube assembly must not be used if symptoms of mechanical, electrical or radiation-related damage are present.

Manipulation of the security and monitoring devices of the unit is prohibited.

The load limit values listed in this operator manual may not be exceeded.

## **Authorized operators**

In normal operation the user has never access to the tube assembly, because the operation is always controlled by the equipment control board.

For this reason, the tube assembly can only be operated by persons authorized to the use of the radiological equipment where the tube assembly is installed, in accordance with the country specific regulations.

The X-ray tube assembly may be replaced only by personnel trained for service on the specific radiological equipment.

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
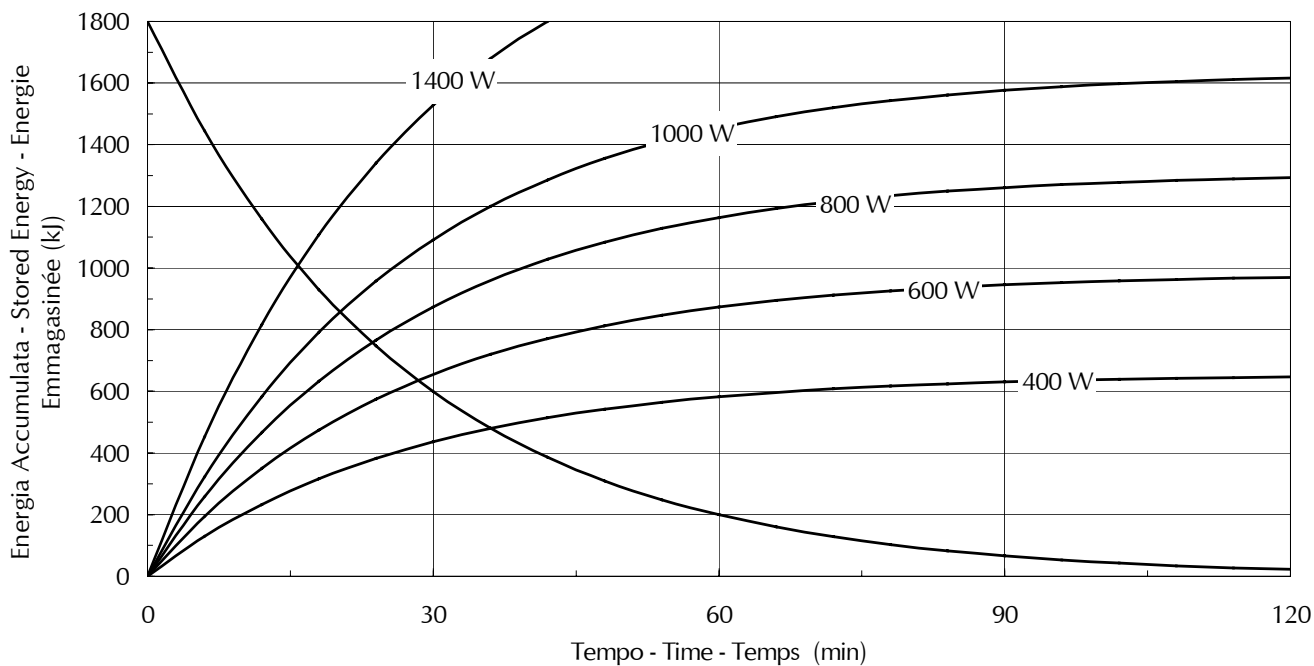
**Ventilatori in funzionamento continuo / Continuously operated fans / Ventilation continue**

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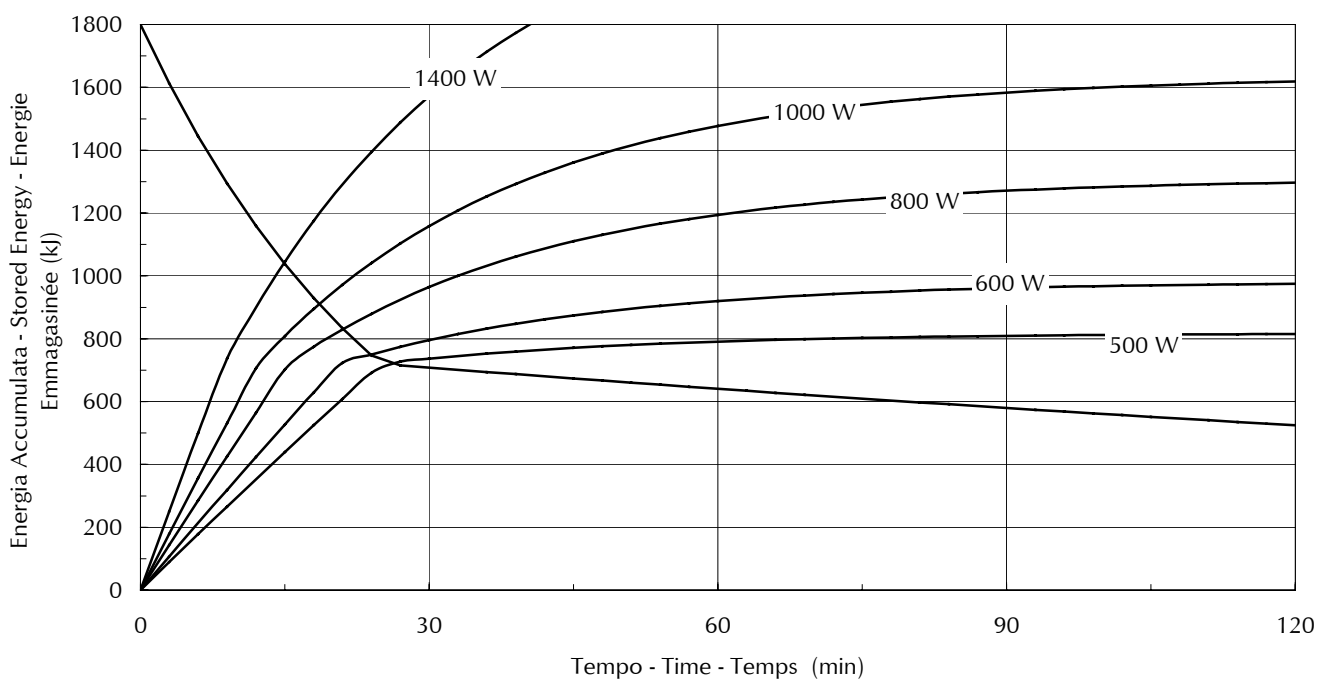## Cooling

Housing cooling is performed by: two internal pumps and two external fans:

- Internal pumps must be on always during tube operation and cooling.
- Fans can be operated continuously with the pumps, or be switched on above 43°C by a thermal switch (see attached schematics). This latter configuration minimizes noise and air flow, but increases housing cooling delays. Maximum cooling rate is not affected. Fan connection can be modified on field. The tube unit is shipped from the factory in the fan continuous operation configuration.
- Operation with pumps and fans not connected is not permitted, except for average dissipation below 100 W.

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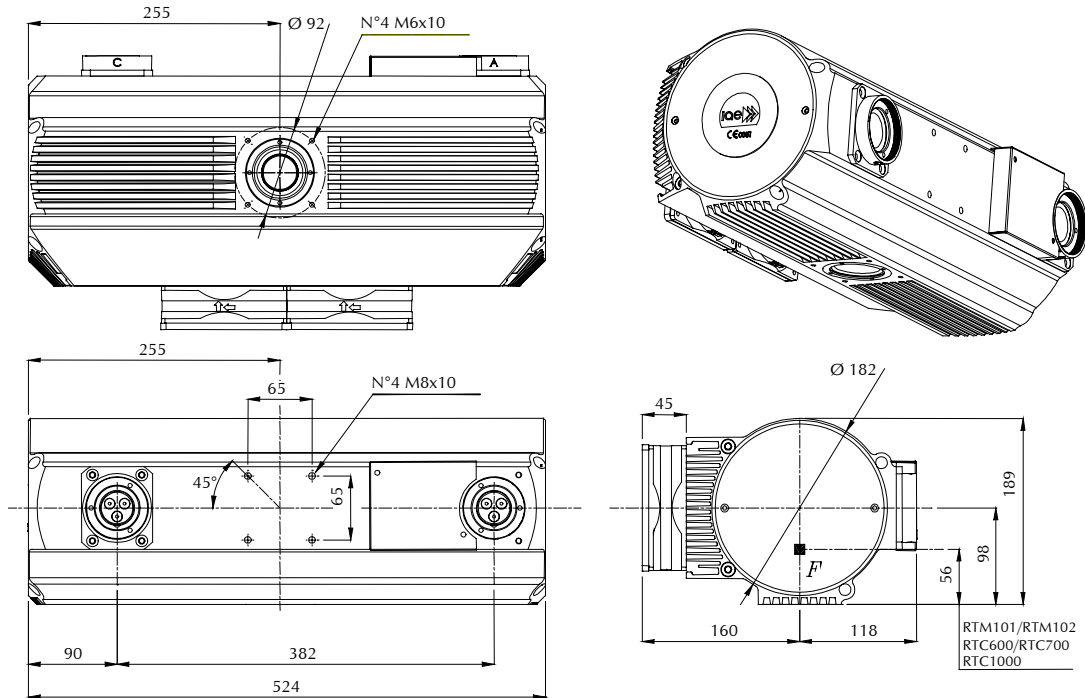
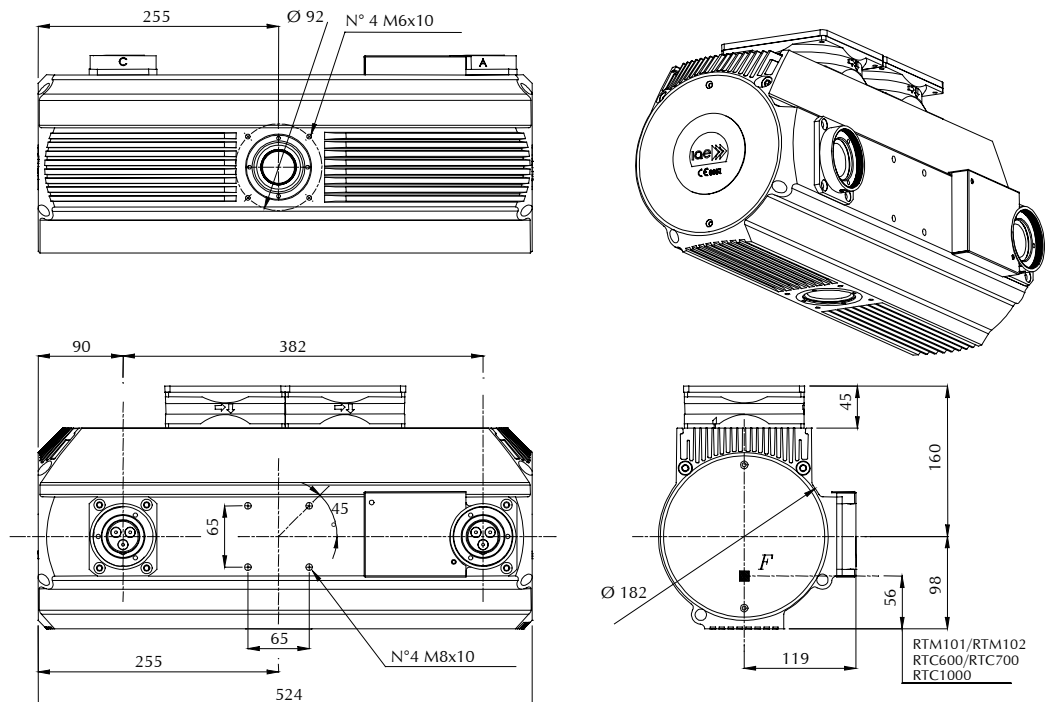
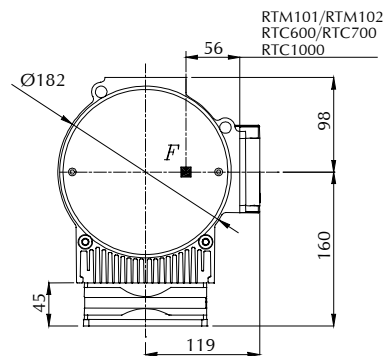
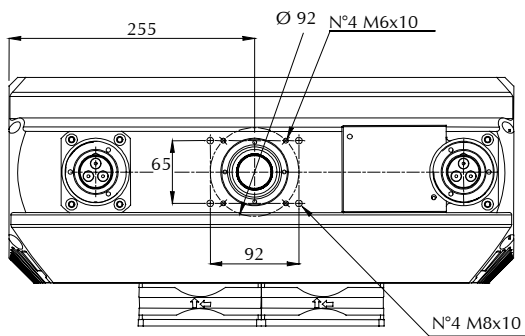
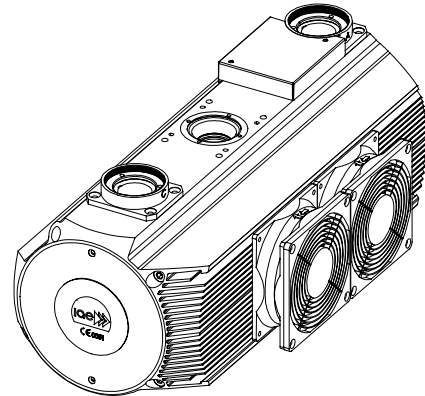
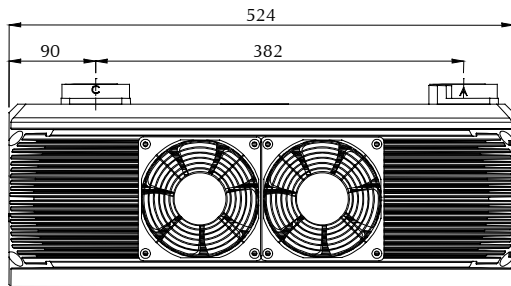
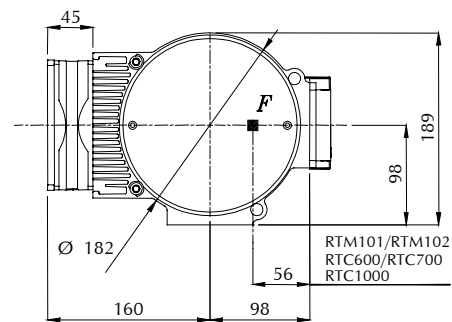
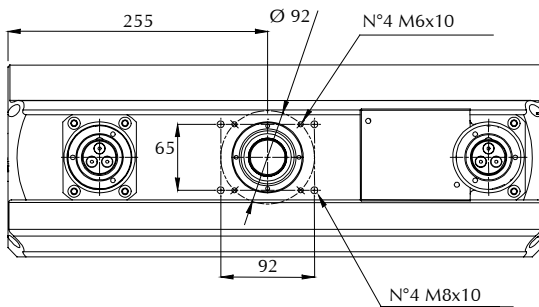
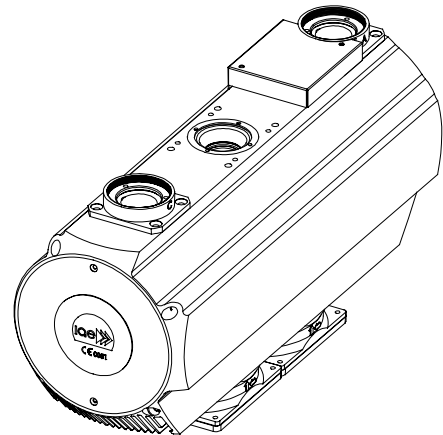
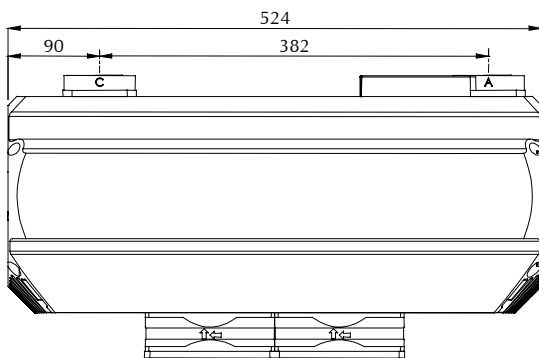
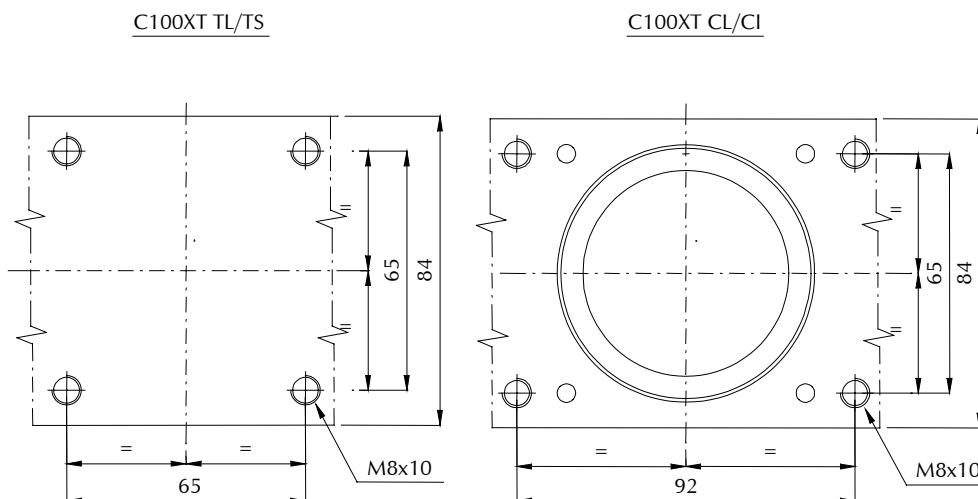
To install the X-ray tube assembly on the equipment we have

#### four threaded holes M8

Minimum number: 4 pcs.  
Minimum strength class: 8.8  
Min engagement length: 9 mm  
Min engagement turns: 7  
Max engagement length: 10 mm  
Max engagement turns: 8  
Max tightening torque: 20 Nm

The screws must be secured against possible disengaging by adequate means, according to the expected vibrations during transportation and operation.

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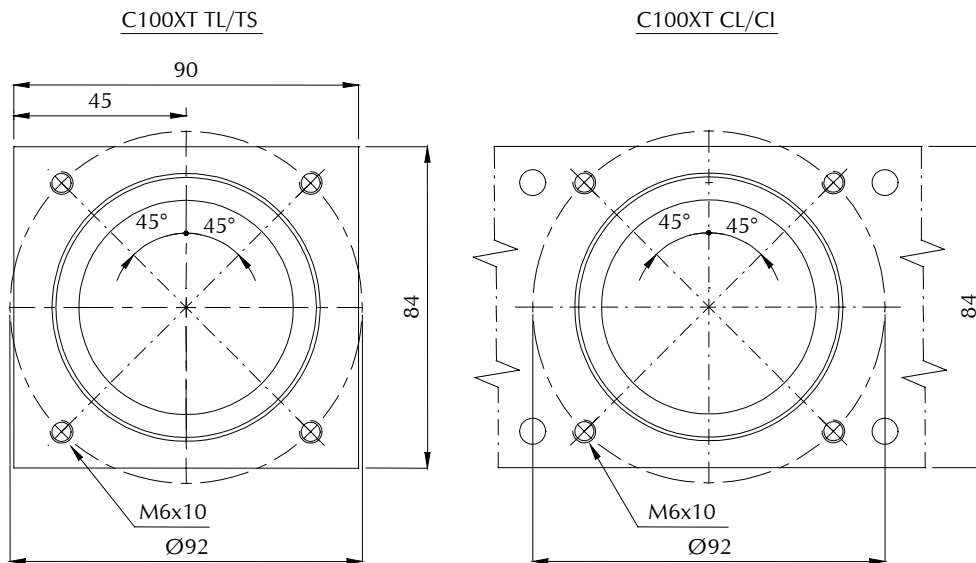
Hazard of loose X-ray tube assembly caused by incorrect tightened screw.  
If you do not obey the above instruction, there is a risk of death or serious injury.

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Beam limiting devices must interface mechanically with the tube unit so that the continuity of leakage radiation shielding is ensured, and must have a leakage radiation low enough to comply IEC 60601-1-3 or other local regulations, when added to the tube unit leakage radiation.

If a structural element is interposed between the housing and the collimator, the shielding must be integrated with suitable parts to ensure continuity of shielding.

Also, longer screws must be used for mounting, in order to obtain the specified screws engaged length. Compliance with requirements of paragraph Mechanical mounting must be ensured.

A maximum mass of 15 kg is admitted for the collimator.

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Prior than inserting the high voltage cables into the cable sockets, both these and the cable terminals must be thoroughly cleaned. Carbon traces from previous arcings, if any, must be completely removed. Cables must be replaced if they are darkened or swollen near the cable terminals, or if these show any crack.

Apply insulating grease or oil, and additional insulating elements if any, according to the instructions of the cables manufacturer.

It is essential that at the end of the cables installation process, no air or contaminating substance remains in the interface between the mated connectors.

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
The X-ray tube assembly must be earthed with a separately routed protective earth (PE) conductor cable, green/yellow.

Connect the PE conductor to the central earthing point of the X-ray tube assembly.

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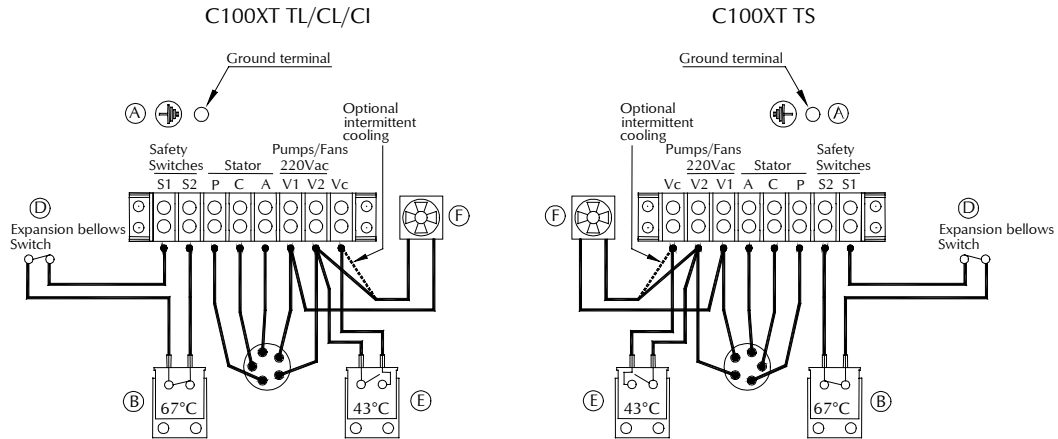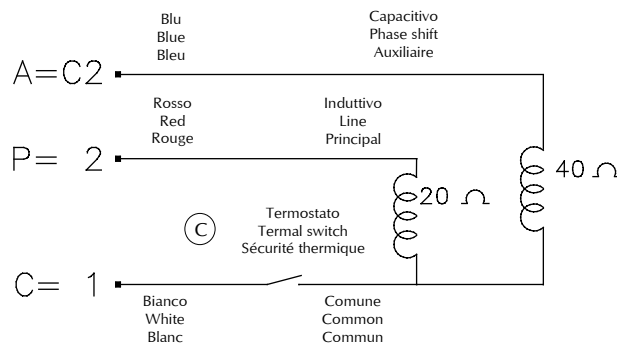
The standard version of the housing is not provided with watertight protection.

If the housing utilization or its position with respect to the patient or the operator makes it possible spilling of water, disinfectant, organic liquids or other conductive liquids over the housing, the same must be protected with a suitable cover to avoid electrical conduction from the patient or operator to the stator power supply and the thermal safeties, which are located below the terminal board cover.

When this situation occurs, it's absolutely necessary to interrupt the housing use and its power supply, until the liquids have been removed.

The stator power supply must be built so that it is isolated from the mains.

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Three devices are present for thermal protection:

- one bimetallic thermal switch, fitted externally on the anode end (switches off at 67 °C);
- one bimetallic thermal switch, internally assembled, series connected with stator common cable (switches off at 80°C).
- a microswitch, installed on cathode end, activated by bellows expansion

The first thermal switch and the microswitch are accessible to the installer. They must be connected to the system safety circuits. These devices are emergency devices. They must not be intended as a normal operation limiting devices. The second thermal switch must be considered as the last safety level; it operates in case of unsuccessful functioning of the other devices.

**INSTALLER MUST ALWAYS CONNECT THE FIRST TWO DEVICES EVEN IF THE LAST ONE IS PRESENT !**

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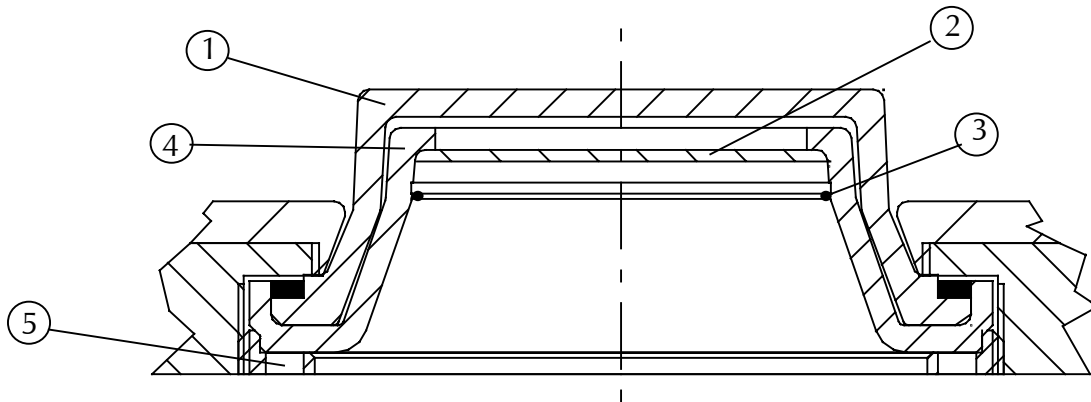
### Warning

The tube assembly is supplied with a 0.3 mm Al filter already installed into the window cone and an additional 1.0 mm Al filter available to the installer.

The tube assembly with the filter already installed complies with the regulations concerning filtration (IEC 601.1.3) (total filtration 1.5 mm Al).

It is the responsibility of the installer to take care that the radiation source assembly complies with the pertinent regulations (total filtration 2.5 mm Al).

If needed the supplied additional filter may be used.

*Note: to install the additional filter, first remove the fastening spring 3.*

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